How to use

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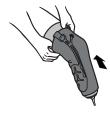
Watch the vide

Position the Overlay with the air system behind the limb. If you feel tibial pressure, rotate it slightly until comfortable.

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Slide the Overlay over the liner ensuring it sit 4cm above distal end.

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The prosthesis can be slid back on. All the air cells must be in the socket below the trim line

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Now you are in control! Simply deflate the Overlay before removing the prosthesis and the liner.

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Use the integrated pump to fine-tune cushioning throughout the day.

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Press the silver valve pin to deflate the Overlay.

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Remove the prosthesis and the Overlay.

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Stretch and press the air system while activating the exit valve to release remaining air.

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Washing Precautions

- Do not put in the washing machine and do not wring out the device to avoid damage.
- O Do not expose the integrated pump and valve to water or fluid.
- O Do not place it in a dryer or use any other heat source for drying.
- The use of unsuitable detergents may cause the device to deteriorate and void the warranty
- → Clean the device once a day only with approved cleansing products including neutral soap or mild detergent that is pH balanced

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